

# Onli

Ready to Use Hydrophilic  
Intermittent Catheter



Size	16 in	7 in	Box Qty
8 Fr	82084-30	82081-30	30
10 Fr	82104-30	82101-30	30
12 Fr	82124-30	82121-30	30
14 Fr	82144-30	82141-30	30
16 Fr	82164-30		30

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

- Ready to use, single-use intermittent catheter
- Hydrophilic-coated, phthalate-free catheter
- 2 smooth catheter eyelets
- Color-coded funnel
- Not made with natural rubber latex
- Gripper on 16-inch catheter to aid insertion

**WARNING:** To help reduce the potential for infection and/or other complications, do not reuse.

**WARNING:** If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

**PRECAUTION:** Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra • Unexplained urethral bleeding • Pronounced stricture • False passage
- Urethritis – inflammation of the urethra • Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

**PRECAUTION:** Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

**CAUTION: Rx Only.** Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product. Prior to use of this device, be sure to read (i) the complete information on how to use this device including Warnings, Cautions, and Instructions for Use, and (ii) all other packaging inserts and labels supplied with the product.

**NOTE:** Store boxes in a flat position and at normal room temperature.

This product consists of: thermoplastic gripper (16-inch catheter only), polyvinylchloride funnel, and tubing not made with phthalates.

1. Data on file. Loose water in Onli catheters and competitor products packaging study. Hollister, 2015.
2. Data on file. Analysis of potential loose liquid to stain clothing for Onli catheters and competitor products. Hollister, 2015.



The Hollister logo and Onli are  
trademarks of Hollister Incorporated  
© 2017 Hollister Incorporated. 923091-117

Call today for more information or a sample:  
888.740.8999 or visit [hollister.com/onli](http://hollister.com/onli)

# Onli

Ready to Use  
Hydrophilic  
Intermittent Catheter

Whether your  
day is ordinary  
or extraordinary,  
the Onli catheter  
is there for you

Designed to easily  
fit into your daily routine  
as a hassle-free and  
mess-free experience

**Ready. Simple.**

Easy to use  
from the start



# Onli

Ready to Use Hydrophilic  
Intermittent Catheter

Continence Care  
People First



### Ready

Prehydrated and ready to use as soon as the package is opened

### Better control

The ergonomic gripper on the 16-inch catheter helps to keep the catheter from being touched



# Relief you can count on, whatever the day may bring

## The Onli catheter is designed for easy, everyday use

The Onli ready to use hydrophilic intermittent catheter is designed to provide a hassle-free and mess-free experience.

## Onli

Ready to Use Hydrophilic  
Intermittent Catheter



- Open with confidence
- Ready to use with reduced spill<sup>1</sup> and no stain<sup>\*2</sup>
- Smooth hydrophilic coating for smooth insertion and withdrawal

<sup>\*</sup> on denim, linen and polyester



### Simple

Ready to use catheter in easy to open foil packaging

### Smooth, easy insertion

The catheter design provides a balance of flexibility and rigidity to support ease of insertion without touching the catheter

