



CeraPlus™ Ostomy Product* Guide



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches


Paediatric Pouchkins™

Skin Essentials

Resources

Welcome to your Hollister Ostomy Product Guide

Within this guide you will find a range of products designed to suit individual needs, including CeraPlus™ Products, an advanced line of ostomy products that help keep skin healthy.

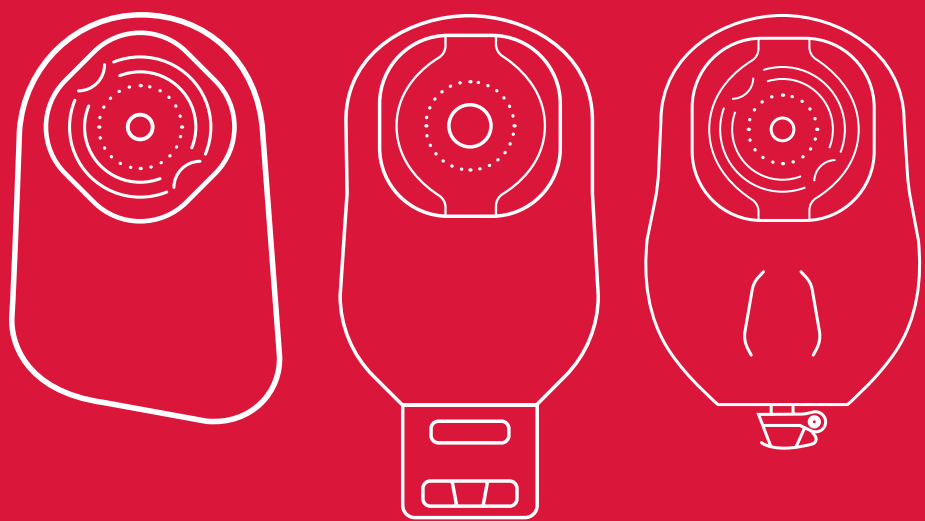

Hollister CeraPlus™ Ostomy Products have received official dermatological accreditation from the Skin Health Alliance, which may bring confidence and reassurance to end users.





Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

CeraPlus™ Ostomy Products

Protection Where it Matters Most

CeraPlus™ Products are demonstrated by evidence to reduce the occurrence and impact of PSCs



As many as **75% of ostomates** have peristomal skin complications (PSCs).¹ CeraPlus™ Products protect skin from Day 1 making it an ideal **first-line choice for your ostomy patients**. When compared to non-CeraPlus™ Skin Barriers, significantly more (75%) patients were “**very satisfied**” with the overall performance of CeraPlus™ Products.²

- Helps support and maintain healthy peristomal skin²
- Delivers positive outcomes for ostomy-related costs and care²
- Provides fit with convexity options and adhesion performance to enhance skin health²



Click here for
Closed Pouches

CeraPlus™ One-Piece Closed Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind
Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*

Comfortable and secure fit
Designed to conform to individual and changeable body shapes for comfort and security against leakage

Built in belt loops
Option to attach a belt for peace of mind and additional pressure if required

Ceramide infused skin barrier
Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²

Integrated soft convex gasket
Provides a secure and comfortable fit whilst applying gentle pressure to help the stoma protrude further into the bag

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Closed Pouches**

CeraPlus™ One-Piece Closed Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	52200	-	30
15-55 mm	52201	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	522100	-	30
15-55 mm	522101	-	30
25 mm	522125	-	30
30 mm	522130	-	30
35 mm	522135	-	30
40 mm	522140	-	30



Click here for more
Closed Pouches

CeraPlus™ One-Piece Closed Pouches



CeraPlus™ Convex Pouches


CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	52600	-	30
15-51 mm	52601	-	30
25 mm	52625	-	30
30 mm	52630	-	30
35 mm	52635	-	30

MAXI	Beige+	Clear	Pack size
15-38 mm	526100	-	30
15-51 mm	526101	-	30



Click here for more
Closed Pouches

CeraPlus™ One-Piece Closed Pouches

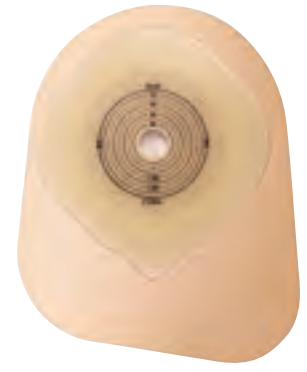

CeraPlus™ Flat Pouches

FLAT			
MIDI	Beige+	Clear	Pack size
15-55 mm	52100	-	30
25 mm	52125	-	30
30 mm	52130	-	30
35 mm	52135	-	30
MAXI	Beige+	Clear	Pack size
15-64 mm	521101	-	30

+Pouch features Viewing Option

When the flap is lifted, the stoma and output can be observed clearly and privately. This feature can also help patients position the pouch more easily, and help maintain their peace of mind.



Click here for **Drainable Pouches**

CeraPlus™ One-Piece Drainable Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind
Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*

Comfortable and secure fit
Designed to conform to individual and changeable body shapes for comfort and security against leakage

Ceramide infused skin barrier
Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²

Built in belt loops
Option to attach a belt for peace of mind and additional pressure if required

Integrated soft convex gasket
Provides a secure and comfortable fit whilst applying gentle pressure to help the stoma protrude further into the bag

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Drainable Pouches**

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	540200	530200	30
15-55 mm	540201	530201	30
25 mm	540225	-	30
29 mm	540229	-	30
32 mm	540232	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	541200	531200	30
15-55 mm	541201	531201	30
25 mm	541225	-	30
29 mm	541229	-	30
32 mm	541232	-	30
35 mm	541235	-	30
38 mm	541238	-	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX with Adhesive Border

MIDI	Beige+	Clear	Pack size
15-25 mm	54002	-	30
15-38 mm	54000	-	30
15-55 mm	54001	-	30
25 mm	54025	-	30
30 mm	54030	-	30
35 mm	54035	-	30

MAXI	Beige+	Clear	Pack size
15-25 mm	54102	-	30
15-38 mm	54100	-	30
15-55 mm	54101	-	30
25 mm	54125	-	30
30 mm	54130	-	30
35 mm	54135	-	30



Click here for more
Drainable Pouches

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Convex Pouches

CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	546200	-	30
15-51 mm	546201	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	548200	-	30
15-51 mm	548201	-	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Convex Pouches

CONVEX with Adhesive Border

MIDI	Beige+	Clear	Pack size
15-25 mm	54602	-	30
15-38 mm	54600	-	30
15-51 mm	54601	-	30
25 mm	54625	-	30
30 mm	54630	-	30
35 mm	54635	-	30

MAXI	Beige+	Clear	Pack size
15-25 mm	54802	-	30
15-38 mm	54800	-	30
15-51 mm	54801	-	30
25 mm	54825	-	30
30 mm	54830	-	30
35 mm	54835	-	30



Click here for more **Drainable Pouches**

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Flat Pouches

FLAT			
MIDI	Beige+	Clear	Pack size
15-64 mm	542200	-	30
MAXI	Beige+	Clear	Pack size
15-64 mm	543200	534200	30



Click here for more
Drainable Pouches

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Flat Pouches

FLAT with Adhesive Border			
MIDI	Beige+	Clear	Pack size
13-64 mm	54200	-	30
25 mm	54225	-	30
30 mm	54230	-	30
35 mm	54235	-	30
MAXI	Beige+	Clear	Pack size
13-64 mm	54300	53400	30
25 mm	54325	-	30
30 mm	54330	-	30
35 mm	54335	-	30



Click here for more
Drainable Pouches Information

CeraPlus™ One-Piece Drainable Pouches



Flextend™ Skin Barrier



High Output with Soft Drain Tap

FLAT with Adhesive Border

MAXI	Beige	Clear	Pack size
13-64 mm	-	26064	10



Lock 'n Roll™

The microseal closure contains over a thousand tiny microseals which securely engage when pressed together, which should create a water, weight, and pressure-resistant seal.



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

CeraPlus™ One-Piece Urostomy Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind

Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*



Built in belt loops

Option to attach a belt for peace of mind and additional pressure if required

Enhanced flow-control tap

The tap is made with softer material and a streamlined shape. The goal of this design? To help increase comfort against the body, so your patients can do more, with less worry.



Multi-chamber design

The pouches have multiple built-in chambers, which are designed to facilitate even distribution of urine. As urine flows into the pouch, it is dispersed among the chambers. This results in a more balanced collection of urine, and a lower pouch profile for your patients.



Comfortable and secure fit

Designed to conform to individual and changeable body shapes for comfort and security against leakage.



Ceramide infused skin barrier

Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²



Integrated soft convex gasket

Provides a secure and comfortable fit whilst applying gentle pressure to help the stoma protrude further into the bag.

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX

	Beige+	Clear	Pack size
15-25 mm	598200	599200	10
15-38 mm	598201	599201	10
20 mm	-	599220	10
25 mm	-	599225	10
30 mm	-	599230	10



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX with Adhesive Border

	Beige+	Clear	Pack size
13-25 mm	-	59900	10
13-38 mm	-	59901	10
13-55 mm	-	59902	10
20 mm	59820	-	10
25 mm	59825	-	10
30 mm	59830	-	10



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Convex Pouches

CONVEX			
	Beige+	Clear	Pack size
15-25 mm	596200	597200	10
15-38 mm	596201	597201	10



CeraPlus™ Convex Pouches

CONVEX with Adhesive Border			
	Beige	Clear	Pack size
13-25 mm	-	59700	10
13-38 mm	-	59701	10
13-51 mm	-	59702	10



Click here for more Urostomy Pouches

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Flat Pouches

FLAT with Adhesive Border

	Beige	Clear	Pack size
13-64 mm	-	59400	10

Urostomy Essentials

	Order No.	Pack size
Urostomy Night Drainage Bag (2000mL)	5550	10
Urostomy Drain Tube Adapter	7331*	10

Urinary Leg Bags

Sterile Leg Bag 500mL 10 cm inlet (regular)	9621	10
Sterile Leg Bag 500mL variable inlet (regular)	9624*	10
Sterile Leg Bag 800mL 10 cm inlet (regular)	9631	10
Sterile Leg Bag 800mL 50 cm variable inlet (regular)	9632*	10

* not available on PCRS



Click here for **Two-Piece Skin Barriers**




CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

CONVEX with Adhesive Border


Stoma	Gasket	Order No.	Pack size
13-25 mm	45 mm	14600	5
13-38 mm	55 mm	15600	5
13-51 mm	70 mm	17600	5
20 mm	45 mm	14620	5
25 mm	45 mm	14625	5
29 mm	55 mm	15629	5
32 mm	55 mm	15632	5

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Skin Barriers

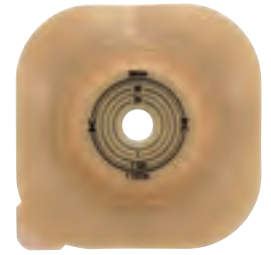


CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

CONVEX

Stoma	Gasket	Order No.	Pack size
13-25 mm	45 mm	14100	5
13-38 mm	55 mm	15100	5
13-51 mm	70 mm	17100	5

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Skin Barriers

CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

FLAT with Adhesive Border

Stoma	Gasket	Order No.	Pack size
13-32 mm	45 mm	14500	5
13-44 mm	55 mm	15500	5
13-57 mm	70 mm	17500	5



CeraPlus™ Skin Barrier

FLAT

Stoma	Gasket	Order No.	Pack size
13-32 mm	45 mm	14200	5
13-44 mm	55 mm	15200	5
13-57 mm	70 mm	17200	5



Click here for
Two-Piece Conform 2 Pouches

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Two-Piece Conform 2 Pouches



Closed Pouch

CLOSED Pouch			
MIDI	Beige	Clear	Pack size
45 mm	24400	24500	30
55 mm	25400	25500	30
70 mm	27400	27500	30
MAXI	Beige	Clear	Pack size
45 mm	24420	-	30
55 mm	25420	-	30
70 mm	27420	-	30

- 45 mm Gasket
- 55 mm Gasket
- 70 mm Gasket






Click here for more
Two-Piece Conform 2 Pouches

Two-Piece Conform 2 Pouches



Drainable Pouch

DRAINABLE Pouch			
MINI	Beige	Clear	Pack size
45 mm	24743	-	30
55 mm	25743	-	30
MIDI	Beige	Clear	Pack size
45 mm	24750	24850	-
55 mm	25750	25850	30
MAXI	Beige	Clear	Pack size
45 mm	24760	24860	30
55 mm	25760	25860	30
70 mm	27760	27860	30

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Conform 2 Pouches

Two-Piece Conform 2 Pouches



Urostomy Pouch

UROSTOMY Pouch with Soft Tap			
MIDI	Beige	Clear	Pack size
45 mm	24790	24890	10
55 mm	25790	25890	10
70 mm	-	27890	10

- 45 mm Gasket
- 55 mm Gasket
- 70 mm Gasket



Click here for Paediatric Pouchkins

Paediatric Pouchkins™



Premature Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Ultra clear one-piece pouch with barrier CTF to 15 mm</p> <ul style="list-style-type: none"> • 50 mm round barrier footprint • No starter hole • With soft wire closure if required 	50	No	3777	15



Newborn Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Ultra clear one-piece pouch with barrier CTF to 35 mm x 22 mm</p> <ul style="list-style-type: none"> • 64 mm round barrier footprint • No starter hole • With soft wire closure if required 	190	No	3778	15



Click here for more
Paediatric Pouchkins

Paediatric Pouchkins™



Child Drainable Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>One-piece clear drainable pouch with Lock'n Roll™ CTF to 50 mm</p> <ul style="list-style-type: none"> • 100 mm triangular barrier footprint • No starter hole 	280	Yes	3795	10



Belts

	Order No.	Pack size
Adapt™ Pouchkins™ Belt (25-43 cm)	3774	1



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



Infant Drainable Pouch

One-piece clear drainable pouch with Lock'n Roll™ CTF to 38 mm

- 70 mm square barrier footprint
- No starter hole

	ISO Vol. (ml)	Filter	Order No.	Pack size
	230	No	3796	10



Infant Urostomy Pouch

One-piece clear urostomy pouch CTF to 38 mm

- 70 mm square barrier footprint
- No starter hole

	ISO Vol. (ml)	Filter	Order No.	Pack size
	230	No	3797	10



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



Infant Drainable Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
Two-piece clear drainable pouch with Lock'n Roll™ closure <ul style="list-style-type: none"> • 45 mm flange 	230	No	3799	10



Infant Floating Flange

	ISO Vol. (ml)	Filter	Order No.	Pack size
Two-piece barrier flange 45 mm CTF to 32 mm <ul style="list-style-type: none"> • 85 mm square barrier footprint • No starter hole 	n/a	n/a	3761	5



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



SoftFlex™ Skin Barrier Drainable Pouch

MINI	Beige	Pack size
15-55 mm	28100	30
25 mm	28125	30
30 mm	28130	30
35 mm	28135	30



Click here for
Barrier Rings

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

Skin Essentials CeraRing™ Barrier Rings

The confidence of a secure seal to help prevent leakage

- Infused with ceramide to protect your skin
- Fills dips and creases around your stoma, for a smooth surface to improve barrier fit
- Stretch, mould or stack to customise with confidence
- Alcohol free, no sting



[Click here](#) for more info or to order a sample



[Click here](#) for more **Barrier Rings**

Skin Essentials CeraRing™ Barrier Rings



CeraRing™ Barrier Rings

	Order No.	Pack size
CeraRing™ Barrier Ring 18 mm	8805	10
Slim CeraRing™ Barrier Ring 18 mm	8815	10
Slim CeraRing™ Barrier Ring 27 mm	881527	10



CeraRing™ Convex Barrier Rings

CeraRing™ Convex Barrier Ring 20 mm	89520	10
CeraRing™ Convex Barrier Ring 30 mm	89530	10
CeraRing™ Convex Barrier Ring 40 mm	89540	10
CeraRing™ Oval Convex Barrier Ring 22-38 mm	89601	10
CeraRing™ Oval Convex Barrier Ring 30-48 mm	89602	10
CeraRing™ Oval Convex Barrier Ring 38-56 mm	89603	10



Click here for more
info or to order a sample



Click here for more
Barrier Rings

Using CeraRing™ Barrier Rings

Option 1:

Apply flat or convex CeraRing™ Barrier Ring to the adhesive side of your skin barrier.



Stretch and shape the barrier ring for a customised fit.

Option 2:

Apply flat or convex CeraRing™ Barrier Ring to your skin, then apply your pouching system. Make sure the barrier ring fits where the skin and stoma meet.



 [Click here](#) for more info or to order a sample

 [Click here for Remover Spray](#)

Skin Essentials Adapt™ Remover Spray

The confidence of an easy pouch removal
that is gentle on skin

Using Adhesive Remover Spray

1. Shake well before use.
2. Peel back one edge of skin barrier.
3. Hold the can so arrow faces the base of the skin barrier, and spray until the adhesive releases.
4. Ensure skin is dry and apply new pouching system as usual.

Spray until the skin barrier releases

- Gently removes adhesive products and residue to make removal of skin barriers fast and easy
- Help reduce potential for skin stripping



[Click here](#) for more
info or to order a sample



[Click here](#) for more
Remover Spray

Skin Essentials Adapt™ Remover Spray



Adhesive Remover Spray

	Order No.	Pack size
Adapt™ No Sting Medical Adhesive Remover Aerosol (50mL)	7737	1



Click here for more info or to order a sample



Click here for **Barrier Extenders**

Skin Essentials CeraPlus™ Barrier Extenders

The confidence of enhancing your pouch security

Enhance the feeling of pouch security by adding a frame to your skin barrier, also helping to extend pouch wear-time.




Increase the feeling of security when undertaking physical activity



Suitable for use in warm climates to reduce barrier edge lifting

Using CeraPlus™ Barrier Extenders

1. Remove existing skin barrier and clean the skin around the stoma. Apply new skin barrier.
2. Remove paper backing by stretching lightly at each end of the barrier extender.
3. Release only one side of the paper backing.
4. Place the adhesive end of the barrier extender along the perimeter of the skin barrier; covering both the skin barrier and the skin next to it. Remove remaining paper backing.
5. Repeat for any additional strips you require around your barrier.
6. Barrier extenders will remove easily with your routine pouch change.

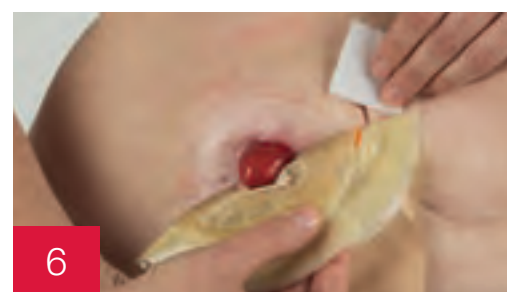

Click here for more info or to order a sample



Click here for more **Barrier Extenders**

Skin Essentials CeraPlus™ Barrier Extenders



Barrier Extenders

	Order No.	Pack size
CeraPlus™ Barrier Extenders (3 sections of hydrocolloid material)	79402	30



Click here for more info or to order a sample



Click here for **Lubricating Deodorant**

Skin Essentials Adapt™ Lubricating Deodorant

The confidence of eliminating odour
and easy pouch emptying

Using in Drainable Pouches

1. Apply pouch.
2. Bottle: add approximately 5ml or 1tsp (Figure 1).
Sachet: add the complete contents of the packet (Figure 2).
Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).
Repeat steps 1 or 2 after every emptying.



Using in Closed Pouches

1. Do not remove release paper on barrier to add product.
2. Bottle: add approximately 5ml or 1tsp (Figure 4).
Sachet: add the complete contents of the packet.
3. Apply your pouch. Rub to thoroughly coat the inside of the pouch (Figure 3).



Lubricate the inside of the pouch to help limit pancaking
and for easy emptying.



[Click here](#) for more
info or to order a sample



[Click here](#) for more
Lubricating Deodorant

Skin Essentials Adapt™ Lubricating Deodorant



Deodorant

	Order No.	Pack size
Adapt™ Lubricating Deodorant Bottle (236mL)	78500	1
Adapt™ Lubricating Deodorant Sachets (8mL each)	78501	50



Click here for more info or to order a sample



Click here for **Skin Essentials**

Skin Essentials



Belts

	Order No.	Pack size
Adapt™ Belt Small/Medium (58-109 cm)	7300	1
Adapt™ Belt Large (74-124 cm)	7299	1
Adapt™ Pouchkins™ Belt (25-43 cm)	3774	1



Powder

Adapt™ Stoma Powder (28g)	7906	1
---------------------------	------	---



Paste

Adapt™ Paste (60g)	79300	1
--------------------	-------	---



Click here for more
Skin Essentials

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

Skin Essentials



Night Drainage Bags 2000mL

	Order No.	Pack size
Night Drainage Bag (Non-Drainable)	9651	10
Night Drainage Bag (Drainable)	9650	10



Urostomy Essentials

Urostomy Night Drainage Bag (2000mL)	5550	10
Urostomy Drain Tube Adapter	7331*	10

* not available on PCRS



Click here for more
Skin Essentials

Skin Essentials



Urinary Leg Bags

	Order No.	Pack size
Sterile Leg Bag 500mL 10 cm inlet (regular)	9621	10
Sterile Leg Bag 500mL variable inlet (regular)	9624*	10
Sterile Leg Bag 800mL 10 cm inlet (regular)	9631	10
Sterile Leg Bag 800mL 50 cm variable inlet (regular)	9632*	10

CeraRing™ Barrier Rings

CeraRing™ Barrier Ring 18 mm	8805	10
Slim CeraRing™ Barrier Ring 18 mm	8815	10
Slim CeraRing™ Barrier Ring 27 mm	881527	10

* not available on PCRS



Click here for more
Skin Essentials

Skin Essentials

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

CeraRing™ Convex Barrier Rings

	Order No.	Pack size
CeraRing™ Convex Barrier Ring 20 mm	89520	10
CeraRing™ Convex Barrier Ring 30 mm	89530	10
CeraRing™ Convex Barrier Ring 40 mm	89540	10
CeraRing™ Oval Convex Barrier Ring 22-38 mm	89601	10
CeraRing™ Oval Convex Barrier Ring 30-48 mm	89602	10
CeraRing™ Oval Convex Barrier Ring 38-56 mm	89603	10



Barrier Rings

Adapt™ Barrier Ring Flextend M™ (98 mm)	7806	10
--	------	----



Click here for more
Skin Essentials

Skin Essentials

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Barrier Extenders

	Order No.	Pack size
CeraPlus™ Barrier Extenders (3 sections of hydrocolloid material)	79402	30

Miscellaneous

Universal Catheter Access Port (Sterile)	9779*	8
--	-------	---

* not available on PCRS





Click here for Resources


Resources

We are here to help with information and education so everyone can live their best life with an ostomy.

Click on one of the booklets to view or download

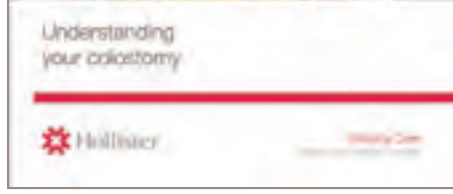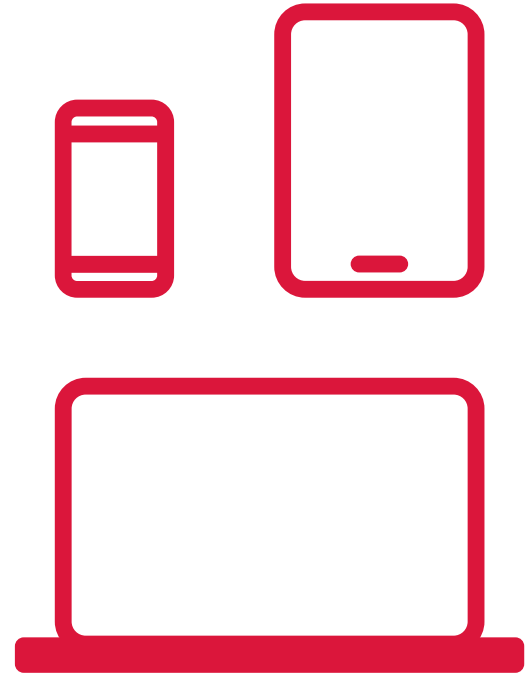


Understanding your Colostomy

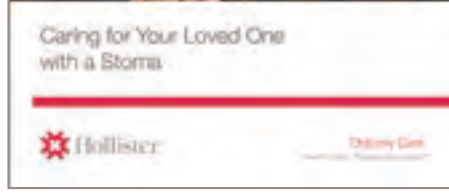

Caring for Your Loved One with a Stoma



Understanding your Ileostomy

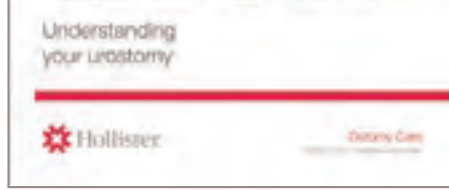

Understanding your Urostomy



Click here for more Resources

Resources



Caring for Your Child with a Stoma



Living with a Stoma: Travel



Living with a Stoma: Healthy Eating



Living with a Stoma: Sex & Parenthood



Living with a Stoma: Home & Work Life



Living with a Stoma: Sports & Fitness



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

For more information or for **FREE** samples, contact us on:

IRE: 1 800 503 400

email: customerservicesie@hollister.com

To find out more about the best ways to maintain healthy skin, visit **www.hollister.ie**

Ireland

Hollister Ltd

Unit 4045

Kingswood Road

Citywest Business Park

D24V06K

www.hollister.ie

References:

1. Smith AJ, Lyon CC, Hart CA. Multidisciplinary care of skin problems in stoma patients. Br J Nurs. 2002 Mar;11(5):324-30.
2. Colwell J, Pittman J, Raizman R, Salvadalena G. A Randomized Controlled Trial Determining Variances in Ostomy Skin Conditions (ADVOCATE). J Wound Ostomy Continence Nurs. 2018;45(1):37-42.

*CeraPlus™ Skin Barriers contain the Remois Technology of Alcare Co., Ltd.

Prior to the use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions. Not all products are CE marked. All other trademarks and copyrights are the property of their respective owners. Person received compensation from Hollister Incorporated. Hollister, the Hollister logo, Adapt, CeraPlus, CeraRing, Flexextend M, Lock 'n Roll, Pouchkins and Soft Flex are trademarks of Hollister Incorporated.

© 2025 Hollister Incorporated.