



Hydrofera Blue Ready

WOUND CARE DRESSING
APPLICATION GUIDE

Ready. Set. Go.

**Simply set in place
and secure;
no hydration required.**

Use When:

- Antibacterial properties are needed to help prevent infection
- Absorption of moderate to heavy levels of exudate is required
- Compression therapy is needed
- Total contact casting is needed

Application:

- Simply place and secure; no hydration required
- Secondary dressing not required
- Tape or wrap required for fixation

Wound Site Preparation



Cleanse the wound bed in accordance with established procedures.

Select Size and Apply



Select the appropriate dressing size to ensure the dressing will cover the entire wound.

Secure



Secure the dressing using a bandage or tape.

Hydrofera Blue Ready foam can be used in conjunction with compression wraps and total contact casting.

Wear Time



Dressing can be left in place up to 7 days, depending on the exudate level and clinical conditions of the wound.

Hydrofera Blue Ready foam Ordering Information

Stock No	Product Size	Quantity/Box
HBRS2520	2.5" x 2.5" (6.4 cm x 6.4 cm)	10
HBRS4520	4" x 5" (10.2 cm x 12.7 cm)	10
HBRS8820	8" x 8" (20.3 cm x 20.3 cm)	10

For more information, see product instructions included with the product.

Rx Caution: Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **1.888.740.8999**. In Canada call **1.800.263.7400**.

The Hollister logo is a trademark of Hollister Incorporated. Hydrofera Blue Ready and the logo are trademarks of Hydrofera, LLC.

Made and printed in the USA.

©2014 Hollister Incorporated.

922498-0214

Manufactured for
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Distributed in Canada by:
Distribué au Canada par:
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3

www.hollisterwoundcare.com

